NL

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EN

HOW TO USE A CONDOM

1. Tear open the packaging along the serrated edge. Be careful, as fingernails or sharp objects can damage the condom.
2. Hold the condom with the rolled edge facing outward, squeeze the reservoir, and place the condom on the tip of the erect penis.
3. Unroll the condom completely down the shaft. If the condom cannot be unrolled, it may be on backwards – discard it and use a new one. Check that it does not slip off or fit too tightly, as this can damage the condom. Remove the condom as soon as possible after ejaculation. Hold the condom at the shaft before withdrawing the penis.
4. Dispose of the foil and used condom in a trash can, not in the toilet.

BEFORE YOU START

Check the expiration date on the packaging before use. If the condom wrapper is obviously damaged, discard the condom and use a new one from an undamaged package. Put the condom on before the penis touches the other person. This helps prevent pregnancy and reduces the risk of contracting sexually transmitted diseases. Remember that no method of contraception is 100% effective in preventing pregnancy, HIV, or sexually transmitted diseases.

CONTRAINDICATIONS

This condom is made of natural rubber latex and may cause allergic reactions, including anaphylactic shock, to people who are allergic to latex.

WARNING - STOP USE:

If you experience discomfort or irritation during use. Use each condom only once. Reuse increases the risk of damage or infection. Condoms can pose a suffocation hazard. Keep out of reach of children. If you use the condom for oral sex first, use a new condom for each subsequent sexual practice. Only use lubricants that are recommended for use with condoms. Oil-based lubricants (petroleum jelly, baby oil, and some vaginal suppositories) and some medications for external use that are applied to the penis or vagina can damage condoms. For anal sex, use additional lubricant on the outside of the condom.

WHEN ADVICE IS NEEDED

Talk to a doctor or pharmacist if you are unsure whether you have a latex allergy, if irritation or discomfort occurs after use, or if you are using medications that are applied locally to the penis or vagina. If a condom leaks or breaks during sex, consult a doctor or pharmacist as soon as possible (within 72 hours at the latest).

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